



CPC December 2024 Newsletter

Seal the deal for patient care!
Why CPC uses large self-seal transport bags.....

In preparation for specimen pickups, all specimens collected at your practice should be placed in the large transport bag provided by CPC. All of the individually bagged specimens for each patient should be placed inside and the bag sealed before placing it in a lockbox or handing it to the courier. Transport bags greatly reduce the possibility of a specimen being left behind. The transport bag is also used by some practices to collect patient specimens throughout the day to ensure no specimens are left behind. The use of our transport bag is recommended in place of using rubber bands or staples to hold individual bags together or the use of unlabeled, generic grocery bags and other non-laboratory bags for transport.

*** CPC can also provide labels for the outside of the transport bag, which will indicate how any jars are sealed inside ***


SPECIMEN SUBMISSION FROM
[Your practice name]
[Your practice address]

Number of Jars _____ |



15x 17 transport bag.



Smaller bags placed inside

And, as a CAP accredited laboratory, CPC requires two unique patient identifiers on each specimen as well as on each requisition. Specimen labels should include at least the following information:

- Patient name
- Patient DOB
- Biopsy site

CPC can provide sheets of jar labels for your practice to use, to aid in labeling jars.

Patient: _____
DOB: _____
DOS: _____
BX site: _____

Read more about us on our website - www.cpcpathology.org

.....